

PERFECT™

DESENSITIZER



Indications

B.J.M. Perfect is a desensitizer used for treatment of dentin hypersensitivity. This desensitizer quickly and effectively reduce pain associated with dentin hypersensitivity.

It is used for:

- Prevent pain in exposed dentin areas not requiring restorations
- Prevent hypersensitivity in dentin after preparation of teeth for following indirect restoration

Properties

B.J.M. Perfect is a one bottle liquid material for dentin hypersensitivity treatment. This desensitizer prevents intra-tubular fluid movement and reduces dental permeability by occlusion of the peripheral dentinal tubules. The effect occurs immediately, effectively, simply and fast with no need for mixing or curing.

Scientific Papers

1. Clinical evaluation by the Dental Advisor, September 2017.

Technical Data

- Perfect containing purified water, organic aldehyde compound, methacrylate based monomer
- Shelf Life 2 Years

Packaging & Order Information

- **Item #400150** - 1 Bottle 5ml of Perfect Desensitizer
- **Item #400151** - 1 Syringe 1.2 ml of Perfect Desensitizer
4 Delivery Tips

Related Products

Perfect can be used with all B.J.M. adhesives and restoration materials and in every treatment case.

Perfect Desensitizer

CLINICAL EVALUATION

++++

www.dentaladvisor.com

RATING SYSTEM: Excellent + + + + + Very Good + + + + Good + + + +

86% overall rating

B.J.M. Laboratories

www.bjmlabs.com

Description

Perfect Desensitizer is an in-office topical desensitizing liquid that occludes the dentinal tubules. It is applied to a clean surface after first placing a rubber dam. After application, the liquid should be left on the surface undisturbed for 30 to 60 seconds, before gently drying it using compressed air from the air/water syringe tip. After drying, the area should be rinsed. Contact with skin, mucosa, pulpal tissue and eyes must be avoided. **Perfect Desensitizer** is available in 10 mL bottles, has a 24-month shelf life from the date of manufacture and requires refrigeration for storage.

Indications

- In-office desensitizer for exposed cervical areas.
- In-office desensitizer for preparations prior to placement of direct and indirect restorations.

Unique Features

- Can be used as a desensitizer under direct and indirect restorations, and on exposed cervical areas.
- Contains HEMA and glutaraldehyde.

Consultants' Comments

- "Desensitizes under composite and amalgam."
- "Easy to apply, not messy, and I like that it's a single bottle."
- "It works most of the time."
- "Initial relief was good but seemed to wear off after about a week."
- "Effectiveness wears off especially with patients that brush rather vigorously."
- "The necessity to use a rubber dam is very inconvenient."
- "I had difficulty dealing with sensitive root structure while needing to use a rubber dam and avoid gingival contact. The instructions stress no contact with skin or mucosa."

"WORKED VERY WELL IMMEDIATELY ON EVEN MY MOST SENSITIVE PATIENTS."


Evaluation Highlights

Perfect Desensitizer was evaluated by 29 consultants, with a total of 457 uses.


- In-office use.
- Easy dispensing and application.
- Occludes dentinal tubules.
- Immediate relief.
- Requires use of a rubber dam.
- Versatile – desensitizes cervical areas and teeth after preparation.




Key Features:

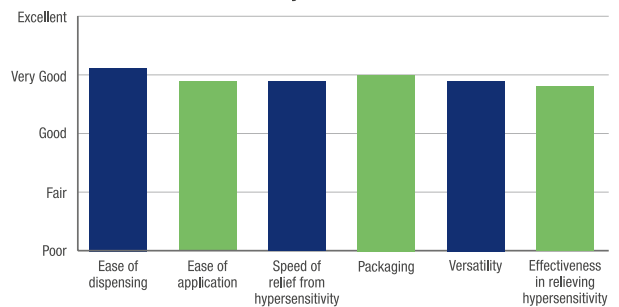

Compared to Competitive Products:

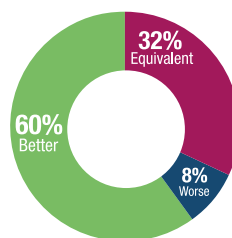

Percentage of Consultants Who Would:

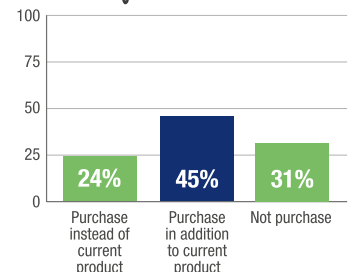

www.dentaladvisor.com

THE DENTAL ADVISOR 3110 West Liberty, Ann Arbor, Michigan 48103 • (800) 347-1330 • connect@dentaladvisor.com • © 2017 Dental Consultants, Inc.

