Auto-Cure ActivatorTM

EN CHEMICAL ACTIVATOR FOR USE WITH PRIMA 2000 OR PRIMA QUICK BOND

INDICATIONS

Auto-Cure Activator is used for self-curing of Prima 2000 in:

- Adhesive cementation of crown & bridge including inlays and onlays.
- Bonding amalgam restorations.
- In light-inaccessible procedures.

INSTRUCTIONS FOR USE

1. Dispense equal number of drops of Prima 2000 and Auto-Cure Activator in a clean mixing well. Mix for 2 seconds with a clean, unused brush or microbrush.
2. Using the same brush, apply ample amount of the mixture and leave undisturbed for 15 seconds.
3. Remove solvents with a vigorous blast of air for at least 15 seconds or until no visible movement of the resin is noted. If the

surface appears incompletely coated, apply additional material and dry again.

4. Finish restoration or cementation.

CAUTION!

Auto-Cure Activator contains acetone and polymerizable monomers which may cause allergic contact dermatitis in susceptible persons. Avoid contact with eyes and skin. Wash thoroughly with water after contact. If skin sensitization occurs, or if known allergy to methacrylate resin exists, discontinue use.

ESSENTIAL INGREDIENTS

- Acetone.
- Ethanol.

- Benzoyl Peroxide.
- Multifunctional monomers.

STORAGE

- Do not store above 24°C (75°F).
- Keep away from direct sunlight.
- For optimum freshness, keep refrigerated.
- Do not freeze.
- Tightly recap after use.
- To obtain SDS visit www.bjmlabs.com.
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.

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