PRIMA Quick™

EN UNIVERSAL, NO-RINSE, SELF-PRIMING ETCHANT ADHESIVE SYSTEM

INDICATIONS

- Direct composite restorations.
- Adhesive cementation of crowns & bridges, including inlays and onlays.

PROPERTIES

• Prima Quick is a sixth generation self-etching bonding system, which delivers high bonding values to both, dentin and enamel. • Prima Quick consists of an acidic water base self-etching Primer and a light-cured Adhesive. • It is used to bond composite resin filling material, to tooth structure, any self-cure or visible light-cured Composite, Compomer restorative material and with Composite Resin Cements. • Prima Quick is utilizing a moderately acidic Primer. Unlike total-etch, total-bond techniques, the self-etching primer does not open the dentinal tubules completely. The smear layer is solubilized, and due to the high hydrophilic properties of the Primer, it is then able to penetrate into the tubules and peritubular dentin, forming resin tags. In the case of enamel, the

Primer produces a significant pattern with enhanced surface area, leading to improved enamel bonding.

INSTRUCTIONS FOR USE

Application of Prima Quick Prime

1. Apply ample amount of Prima Quick Prime on the prepared, air-dried or blotted tooth surface. Leave undisturbed for 15 seconds.
2. Remove solvents with a blast of air for at least 10 seconds.

Application of Prima Quick Bond

3. Apply ample amount of Prima Quick Bond.
4. Remove solvents with a blast of air for at least 10 seconds or until no visible movement of the resin is noted. If the surface appears incompletely coated, apply additional material.
5. Cure for 20 seconds with a visible light cure unit.
6. When the restoration retention area lies primarily within dentin, apply a second layer of Prima Quick Bond. Remove solvents with a blast of air for at least 10 seconds. Do not cure.

7. When light curing is not an option (bonding in root canal), add equal drops of Auto Cure Activator with Prima Quick Bond in a clean mixing well, combine for 2 seconds and proceed with steps 3, 4 and 7.
8. Restore with direct composite resin of choice. For indirect restorations or amalgam bonding, use Prima Quick in conjunction with High-Q-Bond Adhesive Resin Cement.

- Photoinitiators in Bond.
- Acetone.
- Ethanol.
- Adhesive accelerator.
- Water (in Primer).

STORAGE

- Do not store above 24°C (75°F).
- Keep away from direct sunlight.
- For optimum freshness, keep refrigerated.
- Do not freeze.
- Tightly recap after use.
- To obtain SDS visit www.bjmlabs.com.
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.

CAUTION!

Prima Quick contains polymerizable monomers which may cause allergic contact dermatitis in susceptible individuals. Wash thoroughly with soap and water after contact. If skin sensitization occurs, or if you know allergy to methacrylate resin exists, discontinue use.

ESSENTIAL INGREDIENTS

Prime&Bond:

- Multifunctional methacrylates.
- Multifunctional monomers.
- Urethan dimethacrylate.

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