Q GLASS™

EN GLASS IONOMER CEMENT

PROPERTIES

- Definitive sealing of inlays, crowns and bridges and orthodontic brackets.
- Dental cement base.

CONTRAINDICATIONS

In the event of an allergic reaction, consult a specialist.

INSTRUCTIONS FOR USE

MIXTURE: Technical information about the different powder / liquid ratios:

Ratio powder / liquid / in g	Working time	Mixing time	Setting time
1,4 / 1	20"	2'30"	3'00"
1,8 / 1	20"	2'20"	2'40"
2,2 / 1	20"	2'10"	2'30"

The recommended dosage in grams is 2.2 / 1, which corresponds to 1 spoon of powder for 3 drops of liquid. For a liquid consistency, use slightly less powder.

- Put the powder and the liquid on the mixing block.
- With a plastic spatula, introduce the powder into the liquid.
- Mix briskly for 20 seconds.

NOTE: for larger quantities, divide the powder into 2 equal parts.

- Mix the first part with all the liquid for 5 seconds.
- Incorporate the second part of powder and mix for 15 seconds.

SEALING TECHNIQUE

- Prepare the tooth according to the usual protocol.
- Clean with water and pumice. Rinse and dry without draining so as not to alter the "smear layer". For pulp capping, use a calcium hydroxide cement.

- Clean and dry the intrados of the prosthetic part.
- Mix the necessary quantity of cement.
- Apply a 1 mm layer of cement to the inside surface of the prosthetic and position it within 30 seconds after mixing has ceased.

Caution: high temperatures reduce working time.

- The setting time is 5 minutes from the beginning of mixing. Once it has hardened, remove the excess material and protect the prosthetic joint with a polish or unfilled polymerisable resin.

CAUTION

If the liquid should come into contact with the tunica mucosa or the skin, eliminate immediately with alcohol-soaked cotton wool. In the event of eye-contact, rinse thoroughly with water. Do not mix the powder and the liquid with another glass ionomer based component.

ESSENTIAL INGREDIENTS

Liquid & Powder:

- Strontium aluminosilicate glass powder.
- Acrylic acid.
- Tartaric Acid.

STORAGE

- Do not store above 24°C (75°F).
- For optimum freshness, keep refrigerated.
- Do not freeze.
- To obtain SDS visit www.bjmlabs.com.
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.

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