# Zirconite™

## DUAL CURED PERMANENT ADHESIVE RESIN CEMENT SPECIALLY FORMULATED FOR ZIRCONIA CEMENTATION

EN

### PROPERTIES

Zirconite is a novel dual cured permanent adhesive resin cement specially formulated for Zirconia cementation

### INSTRUCTIONS FOR USE

1. Remove the provisional restoration.

**Note:** Use only temporary cements that are eugenol - free.

The use of eugenol-containing temporary luting materials can inhibit the polymerization process of Zirconite.

2. Rinse thoroughly.

3. Air dry the preparation (do not desiccate).

4. Try in the restoration to ensure proper fit.

5. Bleed cartridge before initial use. Place mixing tip on Auto-Mix syringe.

6. Dispensing the material can be done as follows:

### INLAY/ONLAY

Mount the supplied intra oral tip onto the mixing tip and dispense the cement directly into the cavity preparation covering all surfaces. Seat the restoration gently onto the preparation allowing the cement to flow from all sides.

### CROWN

Dispense the cement directly into the restoration or tooth preparation covering all surfaces. Seat the restoration gently onto the preparation allowing the cement to flow from all margins. Note: Material working time is 1-2 minutes; Set time is 3 minutes (at oral temperature).

- When the restoration is properly seated, remove all excess cement. Excess cement is best removed in its gel state with a scaler or explorer. Gel state can be achieved by light curing excess with a light for 2 seconds or allowing to self-cure for 1.5-2 minutes after application. For best results, leave undisturbed for 1.5 minutes before light curing.
- After removal of excess cement, light cure all surfaces including margins for 20 seconds.
- Use finishing strips for proximal areas. Polish margins with polishing discs, cups, or points.
- Check occlusion.

### RECOMMENDED CURE TIMES

L.E. Demetron I, 20 seconds or Optilux 501 in Boost mode 20 seconds, Ramp Mode 20 seconds or Regular Mode 20 seconds. For all other lights see manufacturer's recommendation.

### CAUTION!

Uncured methacrylate resin may cause contact dermatitis and damage the pulp. Avoid contact with skin, eyes and soft tissue. Wash thoroughly with water after contact.

**Note:** Automix tips are designed for single patient use only. ☹

### ESSENTIAL INGREDIENTS

#### Base & Catalyst:

- Multifunctional methacrylates.
- Triethyleneglycoldimethacrylate.
- Methacrylated phosphoric acid esters.
- Polymerization activators.
- Fumed Silica.
- Photoinitiators.
- Adhesive accelerator.
- Glass filler.

### STORAGE

- Do not store above 24°C (75°F).
- Keep away from direct sunlight.
- For optimum freshness, keep refrigerated.
- Do not freeze.
- If refrigerated, allow the syringe to reach room temperature.
- To obtain SDS visit [www.bjmlabs.com](http://www.bjmlabs.com).
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.

**B.J.M. Laboratories Ltd.**  
12 Hassadna St., Industrial Park,  
Or-Yehuda 6022011, Israel  
tel. 972-74-7000111 fax. 972-3-7353020  
Info@bjmlabs.com [www.bjmlabs.com](http://www.bjmlabs.com)

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