BJM Root Canal Sealer™

BJM Root Canal Sealer™

Two-paste epoxy-amine self-cure resin root canal sealer

For professional dental use only.

Patient Population: Any human undergoing dental treatment related to the procedures listed below under "Indications". The definitive patient group is diagnosed on a case-by-case basis by a licensed dental professional and there are no exclusions for specific patient groups.

INDICAZIONI

- Permanent obturations of root canals combined with root canal points.

PROPRIETÀ

- BJM Root Canal Sealer has a radiopacity value above 2.5 mm Aluminium. (Radiopacity is defined as the ability of a material to be visible in x-ray photographs).
- Non cytotoxic material.
- Outstanding sealing ability.
- Long term stability.
- Moderate flexibility that prevents cracking of fully cured material.
- Excellent wettability and flow properties.
- Low shrinkage.
- BJM Root Canal Sealer has been on the market since 2010. No complaints of any health-related or chemical performance issues were received.

ATTENZIONE

- Acid may slow epoxy polymerization (setting process). Please make sure to wash the canal with copious amounts of water and dry well prior to use of BJM RCS.
- Under certain storage conditions, BJM RCS Amine Paste may slightly separate. It has been shown that this process does not negatively affect the performance of the mixed product.

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- Hypersensitivity to epoxy resins and amino groups. Do not use in case of patient who has allergenic sensitivity to epoxies and amines.
- Not for temporary filling of root canals. For use only within root canals.

CAUTION!

- In case of contact with eyes or skin: wash immediately with plenty of water. Any excess of material must be removed with cotton applicator and then washed with water followed with suction.
- For external use only! If swallowed, drink plenty of water. If any sickness symptoms seek medical assistance.
- BJM Root Canal Sealer contains materials which: sensitive to light / irritating to eyes / irritating to respiratory system and skin / may cause sensitization by skin contact / toxic to aquatic organisms may cause long-term adverse effects in the aquatic environment. For complete symbols guide see below
- Any serious incident that occurs in relation to this device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established. You may also contact our EU Authorised Representative: Obelis S.A. (contact details below).

COMPOSITION

Essential ingredients – EPOXY	% wt.	Essential ingredients – AMINE	% wt.
Epoxy Resin	<50%	Poly (Oxyethylene, Oxypropylene) Diamine	<50%
Zirconium Oxide	<30%	Quaternum Ammonium Macromolecole	<10%
Fumed Silica	<5%	Zirconium Oxide	<30%
		Fumed Silica	<5%

FILLER CONTENT

The filler system comprises mineral fillers and modified silica: 45-65 weight-% or 10-20 volume-%. The particle size of the filler's ranges: <6 µm.

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Note:

Tips are for single patient use only to prevent cross-contamination between patients.

- Prepare the root canal, clean and dry before use of BJM Root Canal Sealer.
- Automatic (When using a dual-barrel syringe with automix syringe mix tip):** Bleed the syringe before installation of the automix syringe mix tip. Mount a new, automix syringe mix tip onto the dual-barrel syringe and bleed the syringe again. This ensures complete flow of Base and Catalyst components.
Hand Mix (When using a dual-barrel syringe with Detachable Manual Adapter): a) Bleed the syringe before installation of the Manual Adapter. Mount a new manual adaptor onto the dual-barrel syringe and bleed the syringe again. This ensures complete flow of Base and Catalyst components. b) Using a

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- clean spatula or mixing stick, mix the two components into a homogenous paste.

- Application of BJM Root Canal Sealer:

- Standard technique:** BJM Root Canal Sealer is used in combination with gutta-percha points. For filling techniques where most of the canal is obturated by endodontic point material, apply only a light coating of BJM Root Canal Sealer to the canal walls.
- Lateral condensation technique:** For the Master-Point-Technique, select a gutta-percha point (or alternatively a paper point or a reamer) in the size of the last instrument used during apical preparation. Wet the canal walls with BJM Root Canal Sealer through a pumping or simultaneously rotating movement in a counterclockwise direction of the point/reamer. Alternatively, apply BJM Root Canal Sealer onto the tip of a Lentulo spiral. Advance the Lentulo spiral slowly to the apex running at very low speed. Avoid the formation of air bubbles in the material and overfilling of the canal. Withdraw Lentulo very slowly, still running at low speed. Dip point into BJM Root Canal Sealer and insert them into the canal with a slow pumping motion.
- Warm gutta-percha obturation technique:** (in 3 methods)
 - Vertical Compaction by Schilder and Machtout Methodology
Put BJM Root Canal Sealer on Master Cone (taper .04 or .06) and insert it 0.5-1 mm shorter than Working Length (WL) by a pumping and brushing movement. Heated pluggers (about 1500°C) softened and cutting gutta-percha and afterwards Cold Hand plunger gradually compact it until the apical 5 mm plug is formed. Put BJM Root Canal Sealer on the canal walls and fill with injectable gutta-percha.
 - Continuous Wave Condensation by Buchanan
Put BJM Root Canal Sealer on Master Cone (taper .04 or .06) and insert it 0.5-1 mm shorter than Working Length (WL) by a pumping and brushing movement. One continuous movement of heated plunger (1500C-2000C) cutting gutta-percha Cone 5 mm shorter WL. Put BJM Root Canal Sealer on the canal walls and fill with injectable gutta-percha.
 - Combination Technique by Solomonov
Put BJM Root Canal Sealer on Master Cone (taper .02) and insert it to Working Length (WL) by a pumping and brushing movement. Deep Lateral Compaction from WL to the 5 mm from the WL with Ni-Ti spreader No. 25 and accessory gutta-percha points. Heating of gutta-percha points inside the canal up to 1000C. Compaction of the heated gutta-percha by hand pluggers Machtout. Repeat the procedure until the dense obturation of the canal.

- Removal of BJM Root Canal Sealer:**

- The conventional technique of removing the gutta-percha is applicable when BJM Root Canal Sealer is used with gutta-percha points.
- Tightly re-cap syringe after use.
- All the instruments that were in contact with BJM Root Canal Sealer should be cleaned immediately after use with alcohol or acetone.
- Time indications:
 - The setting time depend on the oral temperature.
 - The times shown are based on conditions relevant for practice.

	Hours
Intra-oral setting time	<24

Mixing options:

Note: the setting time complies to the mentioned above range without dependence on the mixing options.

Mixing Options	Ratio
Automix – 1 Automix syringe	1:1 – Equal amount of Amine and Epoxy
Hand Mix – 1 Automix syringe with Detachable Manual Adapter	1:1 – Equal amount of Amine and Epoxy

REMOVAL OF BJM ROOT CANAL SEALER

The conventional technique of removing the gutta-percha is applicable when BJM Root Canal Sealer is used with gutta-percha points.

STORAGE AND DISPOSAL

- Do not store above 24°C (75°F).
- Keep product in original packaging.
- Keep away from direct sunlight.
- For optimum freshness, keep refrigerated.
- Do not freeze.
- If refrigerated, allow the syringe to reach room temperature.
- To obtain SDS or IFU visit www.bjmlabs.com.
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.
- Empty product can be safely disposed after use in an intended regular waste container. The product does not require special disposal instructions or precautions and is harmless to the environment.

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- Application du ciment BJM Root Canal Sealer:

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