High-Q-Bond Band.™

High-Q-Bond Band™

Light-cure band cement

For professional dental use only.

Patient Population: Any human undergoing dental treatment related to the procedures listed below under "Indications". The definitive patient group is diagnosed on a case-by-case basis by a licensed dental professional and there are no exclusions for specific patient groups.

EN INDICATIONS

- Direct bonding of metal orthodontic bands to tooth surfaces.
- Occlusion adjustments to prevent deep bite.
- Occlusal interference with a particular orthodontic device.

PROPERTIES

- High-Q-Bond Band bonds chemically and mechanically to tooth surfaces.
- High-Q-Bond Band is a light-cure material - compatible with all curing units available on the market.
- High-Q-Bond Band depth of cure is >4 mm.
- Glass filled material designed specifically for orthodontic use.
- Excellent flow for filling the gap between the band and the tooth.
- Blue visible color. To see shade guide visit [www.bjmlabs.com](#).
- High-Q-Bond Band has been on the market since 2007 with no changes in formula. No complaints of any health-related or chemical performance issues were received.

CONTRAINDICATIONS

- The use of the medical device is contraindicated if the patient is known to be allergic or sensitive to any of its ingredients.
- Direct contact with eugenol-based products is to be avoided. Eugenol inhibits the setting of resin-based cements.

CAUTION!

- Refrigeration will increase viscosity, making syringing more difficult, and will retard setting time.
- High-Q-Bond Band contains polymerizable monomers, which may cause allergic contact dermatitis in susceptible patients. Wash thoroughly with soap and water after contact. If skin sensitization occurs, or if known allergy to methacrylate resin exists, discontinue use.
- Polymerization may be triggered by ambient lighting or the dental operating light. Tightly recap the syringe after use.
- High-Q-Bond Band contains materials which: sensitive to light / irritating to eyes / harmful if swallowed / irritating to respiratory system and skin / may cause sensitization by skin contact. For complete symbols guide see below.
- Any serious incident that occurs in relation to this device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established. You may also contact our EU Authorised Representative: Obelis S.A. (contact details below).

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Essential ingredients	% wt.	Essential ingredients	% wt.
Triethylenglycoldimethacrylate	<20%	Glass filler	<60%
2-hydroxyethylmethacrylate	<5%	Fumed silica	<5%
Polymerization activators	<1%	Titanium oxide	<5%

FILLER CONTENT

The filler system comprises glass fillers and modified silica:

- 60-70 weight-% or 42-50 volume-%.
- The particle size of the fillers ranges: <6 um.

INSTRUCTIONS FOR USE

Note:

If refrigerated, allow to return to room temperature before use. Refrigeration will increase viscosity, making application more difficult, and will retard setting time.

- Clean teeth with plain pumice using a rotating bristle brush or rubber cup.
- Rinse and air-dry.
- Isolate the bonded surfaces.
- Apply a small amount of High-Q-Bond Band Cement onto the tooth-bearing surfaces of the band.
- Immediately seat the band onto the tooth with a slight rotating motion. A small amount of cement should exude around the entire band periphery. Final adjustment must be made before the cement begins to cure from the ambient light (approximately 60 seconds).
- Light-cure entire bonded tooth surface for 20 seconds from two opposite directions.
- High-Q-Bond Band Cement can be easily wiped away with a cotton roll before it starts setting or can be removed with a scaler after it has set.
- Tightly re-cap syringe after use.

	min:sec
Light curing in a light-wave range of 440 – 470 nm Compatible with all curing units available on the market	00:20

STORAGE AND DISPOSAL

- Do not store above 24°C (75°F).
- Keep product in original packaging.
- Keep away from direct sunlight.
- For optimum freshness, keep refrigerated.
- Do not freeze.
- If refrigerated, allow the syringe to reach room temperature.
- To obtain SDS or IFU visit [www.bjmlabs.com](#).
- Do not use after expiration date. See expiry date on the label.
- Self life is 2 years after the date of manufacturing.
- Empty product can be safely disposed after use in an intended regular waste container. The product does not require special disposal instructions or precautions and is harmless to the environment.

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