# HIGH Q BOND™

## EN DUAL CURED PERMANENT ADHESIVE RESIN CEMENT

### INDICATIONS

- Final cementation of crowns and bridges, inlays and onlays, posts and cores, ceramic crowns and Maryland bridges.
- Veneering of alloys.

### PROPERTIES

- High-Q-Bond Adhesive Resin Cement is a multi-purpose, dual cured, radiopaque, permanent cement
- It forms an excellent bond with base alloys, porcelain, dentin, enamel and with amalgams.
- High-Q-Bond Adhesive Resin Cement is a self cured cement which you can light cure for an immediate set and extra strength at the margins.
- Light curing High-Q-Bond Adhesive Resin

Cement gives your restorations immediate stability, plus easy cleanup of excess material.

- With a simple 10-seconds light precure, you can remove excess cement before the final set.
- High-Q-Bond Adhesive Resin Cement features superior retention and total margin integrity.

### INSTRUCTIONS FOR USE

- Depending on the clinical requirement use the corresponding illustrated card. Set time: Working time at room temperature is at least 2 minutes.
- Final set time at room temperature is approximately 5 minutes.
- Oral set time is approximately 3.5 minutes.
- Light cure for 20 seconds.

### CAUTION!

High-Q-Bond Adhesive Resin Cement contains polymerizable monomers which may cause allergic contact dermatitis in susceptible patients. Wash thoroughly with soap and water after contact. If skin sensitization occurs or if known allergy to methacrylate resin exists, discontinue use.

### ESSENTIAL INGREDIENTS

#### Base & Catlyst:

- 4-Methacryloxyethyltrimellitic anhydride.
- Urethan dimethacrylate.
- Multifunctional monomers.
- Polymerization activators.
- Photoinitiators.
- Glass filler.

### STORAGE

- Do not store above 24°C (75°F).
- Keep away from direct sunlight.
- For optimum freshness, keep refrigerated.
- Do not freeze.
- If refrigerated, allow the syringe to reach room temperature .
- To obtain SDS visit [www.bjmlabs.com](http://www.bjmlabs.com).
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.

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